

ELLIOCATH

Curved Catheter for I.U.I.

(Metal Free)

Product description

- Curved catheter enables an easy passage through the cervical canal.
- Arrows located on the grip of ELLIOCATH® indicate the direction of the curve of the catheter.

When to use?

- Intrauterine insemination.

How to Use?

Before use, wash the catheter carefully with an appropriate sterile culture medium.

Intrauterine Insemination

- Fill ELLIOCATH® (connected to syringe) with the sperm preparation.
- Push ELLIOCATH® into the uterus.
- To facilitate cervical canal passage, push the catheter with a gentle rotational movement.
- Push in the sperm preparation slowly.
- Slowly withdraw the catheter.



Product Code	Product	I.D.	Length	Sterilisation
NS1280545	ELLIOCATH®	1.1 mm	20.5 cm	γ irradiation

Marketed in India by:

BONNE SANTÉ
Translating Technology. Transforming Lives

Manufactured by: Laboratoires Ellios Bio Tek, PARIS, FRANCE.

