

Sperm Perfusion (I.U.I.)

(INSEM.Hystero)

Product description

- An internal PEBD catheter.
- A cone-shaped obturating ring.
- A stiff external tube.
- A two ways stopcock.
- The cone-shaped obturator allows an effective and atraumatic occlusion of external cervical os for all anatomic diameters from 6 mm to 20 mm.
- This device is sterilized by gamma radiation.

When to use?

HysteroGRAPHY.


Hysterosonography.

Sperm perfusion (Intrauterine insemination).

Methylene blue test.

How to use?

- Insert OLIVENNES Cath through the cervix into the uterus and push it till an effective occlusion is obtained.
- Maintaining the catheter in position, inject the preparation slowly (sperm preparation, saline water, contrast medium).
- Shut off the stopcock during the procedure.
- Remove the device slowly on completion of procedure.

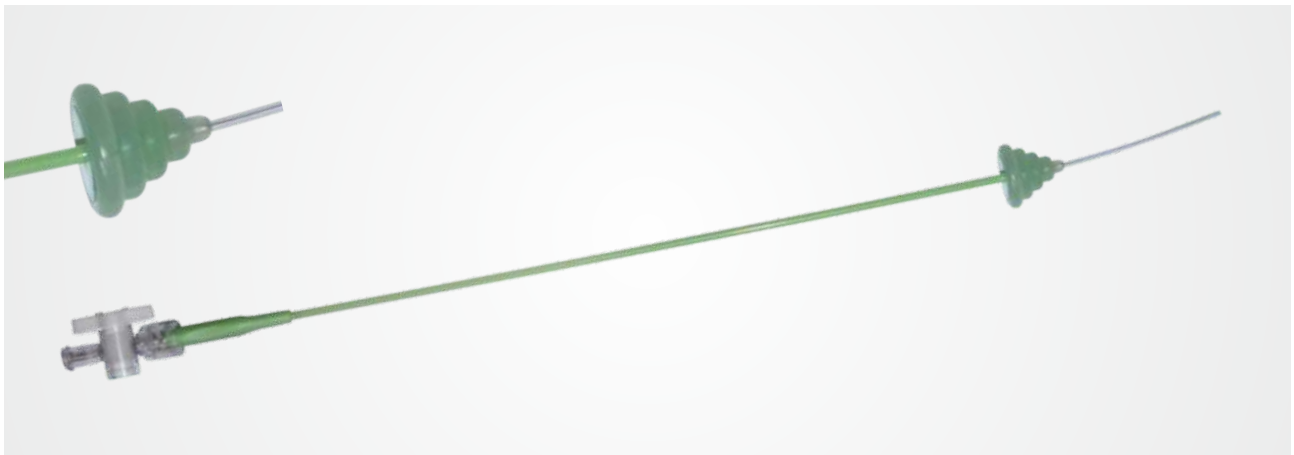

Product Code	Product	Length	Sterilisation
EBT00400-OL	OLIVENNES Cath	273 mm	γ irradiation

Marketed in India by:

BONNE SANTÉ
Translating Technology. Transforming Lives

Manufactured by: Laboratoires Ellios Bio Tek, PARIS, FRANCE.

