

OOCYTES PUNCTURE SET

For 50 ML Falcon Tube

Product description

- Single lumen needle.
- Back-cut bevel (17G / 1.4mm).
- Non-coring bevel (prevents tissue entering lumen).
- 2 laser echo markings.
- Tactile mark indicating bevel position.
- 2 tubes connected to a silicone stopper for 50 ml Falcon-like tube.
- 3 ways stopcock.
- 4 tabs for traceability.

When to use?

- Echo guided oocytes pick up.
- Ovarian cyst aspiration.

How to use?

Before use, wash the needle and the aspiration line carefully with an appropriate sterile culture medium.

- Connect the first falcon type tube to the stopper (to be replaced when needed during the procedure).
- Then aspirate the ovum under controlled pressure.
- Rinse carefully after aspiration (oocytes, if present, will be collected from the needle and the aspiration line).



| Product Code | Product | Specifications | Sterilisation |
|--------------|---|---|---------------|
| EBT00430S-O | Follicule aspiration set (Falcon 50 ml) | Needle: 30 cm - 17 G (1.4 mm) Needle-stopper: 90 cm Stopper-pump: 100 cm | γ irradiation |

Marketed in India by:

BONNE SANTÉ
Translating Technology. Transforming Lives

Manufactured by: Laboratoires Ellios Bio Tek, PARIS, FRANCE.

