

OOCYTES PUNCTURE SET

For Vial sv-XL30

Product description

- Single lumen needle.
- Back-cut bevel (17G / 1.4mm).
- Non-coring bevel (prevents tissue entering lumen).
- Two laser echo markings.
- Tactile mark indicating bevel position.
- Two tubes connected to a screw cap (to be used with a 30 ml vial - sv-XL30).
- Four tabs for traceability.
- Oocytes puncture set with a 3 ways stopcock.


When to use?

- Echo guided oocytes pick up.

How to use?

Before use, wash the needle and the aspiration line carefully with an appropriate sterile culture medium.

- Screw a vial sv-XL30 (to be replaced when needed during the procedure).
- Proceed to aspirate ovum under controlled pressure.
- Rinse carefully after aspiration (oocytes, if present, will be collected from the needle and the aspiration line).
- The oocytes puncture set with a 3 ways stopcock allows intermittent flushing and aspiration.

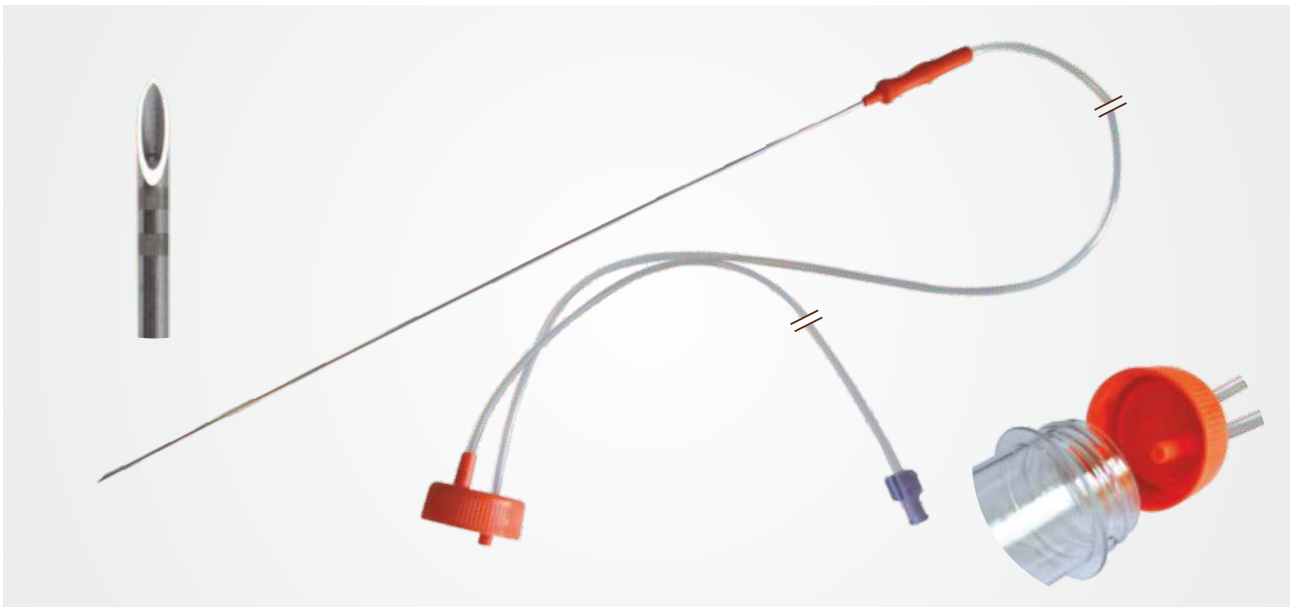

Product Code	Product	Specifications	Sterilisation
EBT00330S-O	Follicule aspiration set with screw cap	Needle: 30 cm - 17 G (1.4 mm) Needle-cap: 90 cm Cap-pump: 100 cm	γ irradiation
EBT00330S-R	Follicule aspiration set with screw cap and with stopcock	Needle: 30 cm - 17 G (1.4 mm) Needle-cap: 90 cm Cap-pump: 100 cm	γ irradiation

Marketed in India by:

BONNE SANTÉ
Translating Technology. Transforming Lives

Manufactured by: Laboratoires Ellios Bio Tek, PARIS, FRANCE.

