

Straight Catheter for I.U.I.

Product description

- Distal P.E. tube with two lateral apertures at distal end.
- Proximal P.P. graduated (cm) tube.

When to use?

- Intrauterine insemination.

How to use?

Before use: Wash the catheter carefully with an appropriate sterile culture medium.

- Fill PMIU connected to the syringe with sperm preparation.
- Push PMIU into the uterus.
- Push the sperm preparation in slowly.
- Withdraw the catheter slowly.



Product Code	Product	Volume	I.D.	O.D.	Length	Sterilisation
PS1260600	PMIU	0.40 ml	1.6 mm	2.1 mm	205 mm	γ irradiation
				6 CH		

Marketed in India by:

BONNE SANTÉ
Translating Technology. Transforming Lives

Manufactured by: Laboratoires Ellios Bio Tek, PARIS, FRANCE.

