

PRELEVINJEC

Device for I.U.I. at Doctor's Office

(Mini syringe for I.U.I.)

Product description

- A P.P. graduated sheath with a small diameter P.E. distal part.
- An acetal resin piston with a P.E. joint.

When to use?

- Intrauterine insemination at doctor's office.

How to use?

Before use, wash the catheter carefully with an appropriate sterile culture medium.

- Fill PRELEVINJEC with the sperm preparation (1 graduation = 100 μ l).
- Insert it slowly into the uterus.
- Slowly push in the sperm preparation.
- Withdraw the mini syringe gradually.



Product Code	Product	I.D.	Graduation	Sterilisation
PS1240623	PRELEVINJEC	2.6 mm	100 μ l	γ irradiation

Marketed in India by:

BONNE SANTÉ
Translating Technology. Transforming Lives

Manufactured by: Laboratoires Ellios Bio Tek, PARIS, FRANCE.

