

## Hysterosonography

### Product description

- An internal PEBD catheter.
- An obturating disk.
- A stiff external tube.
- A two ways stopcock.
- The obturator allows an effective and atraumatic occlusion of external cervical os irrespective of the anatomic diameters.
- This device is sterilized by gamma radiation.

### When to use?

Hysterosonography.

### How to use?

- Insert SONOKAT through the cervix into the uterus and push it till an effective occlusion is obtained.
- Maintaining the catheter in position, inject the preparation slowly (saline water).
- Shut off the stopcock whenever necessary during the related procedure.
- Remove the device slowly after completion of the procedure.



Product Code	Product	Length	Sterilisation
EBT00500	SONOKAT	225 mm	γ irradiation

Marketed in India by:

**BONNE SANTÉ**  
Translating Technology. Transforming Lives

Manufactured by: Laboratoires Ellios Bio Tek, PARIS, FRANCE.

